

# How to apply Pivotal Med Supply Dressings


## COLLAGEN DRESSING



- ✓ Cleanse wound site with gentle cleanser, such as the Pivotal Med Supply wound cleanser
- ✓ Apply Pivotal Med Supply Dressing directly to the wound
- ✓ Apply collagen pad dressing (may be trimmed to size)
- ✓ Cover with non-adherent dressing
- ✓ Repeat process as needed

## COLLAGEN POWDER

### *Dry Application*

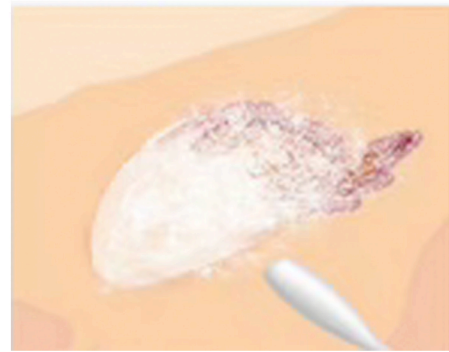

- ✓ Cleanse wound site with gentle cleanser, such as the Pivotal Med Supply wound cleanser
- ✓ Apply Collagen Powder in a layer approximately 1/4 in. thick (do not pack tightly)
- ✓ Apply Pivotal Med Supply Dressing over the powder
- ✓ Cover with non-adherent dressing
- ✓ Repeat process as needed