

# How to Apply

## Pivotal Med Supply Dressings

### Composite Dressings

1

Cleanse wound site with gentle cleanser, such as the Pivotal Med Supply wound cleanser.



Apply Pivotal Med Supply Dressing directly to the wound.

2

3

Apply collagen pad dressing, trimming it to fit the wound if necessary



Cover with non-adherent dressing.

4

**Note:** Repeat process as needed.



2010 E Continental Blvd, Suite F Southlake, TX 76092  
T: 866-510-0040 | F: 833-423-5673 | E: [info@pivotalmedsupply.com](mailto:info@pivotalmedsupply.com)

**PIVOTAL** | MED SUPPLY



# How to Apply

## Pivotal Med Supply Dressings

**Collagen Powder**

Dry Application

1

Cleanse wound site with gentle cleanser, such as the Pivotal Med Supply wound cleanser.



Apply Collagen Powder in a layer approximately 1/4 in. thick (Do not pack tightly)

2

3

Apply Pivotal Med Supply Dressing over the powder



Cover with non-adherent dressing.

4

**Note:** Repeat process as needed.



2010 E Continental Blvd, Suite F Southlake, TX 76092  
T: 866-510-0040 | F: 833-423-5673 | E: [info@pivotalmedsupply.com](mailto:info@pivotalmedsupply.com)

**PIVOTAL** | MED SUPPLY

